



PREVENTIVE MAINTENANCE SCHEDULE **FOR G.E.M. R.O. SYSTEMS WITH CONVENTIONAL PUMPS**

MODEL NO. _____ S/N. _____

DATE ____ / ____ / ____ PERFORMED BY _____

HOUR METER _____ HRS.

PUMP LUBRICATION: - USE SYNTHETIC LUBE ONLY (AMS/OIL) OR EQUIVALENT

☐

TONKAFLO PUMP: - EVERY 4400 HOURS

UPPER FITTING = 6 SHOTS OF LUBE

☐

LOWER FITTING = 6 SHOTS OF LUBE

☐

IF THE PUMP HAS ONLY ONE FLUBE FITTING, INJECT 15 SHOTS OF LUBE

☐

MOTOR LUBRICATION:

5 HP MOTOR AND LARGER

UPPER FITTING - ONE SHOT OF GREASE EVERY 6 MONTHS

☐

LOWER FITTING - (SAME)

☐

MOTORS THAT ARE 3 HP OR SMALLER HAVE SEALED BEARINGS AND DO NOT REQUIRE LUBRICATION.

CHECK FOR FLUID LEAKS

☐

CHECK FOR EXCESSIVE GREASE AT THE BEARING FRAME

☐

REVIEW DAILY LOG SHEET FOR CHANGES IN FLOW, WATER QUALITY, AND MEMBRANE DELTA PRESSURE.

☐