



WATER SOFTENER DISINFECT PROCEDURE

- 1) PREPARE 5 GALLONS OF 10% BLEACH SOLUTION.
- 2) TURN OFF THE WATER TO THE SOFTENER AND CONNECT THE OUTLET TO DRAIN.
- 3) DISCONNECT THE BRINE TUBE FROM THE BRINE TANK.
- 4) INSERT THE BRINE TUBE INTO THE BLEACH SOLUTION.
- 5) ADVANCE THE CONTROLLER TO THE BRINE DRAW CYCLE.
- 6) TURN OFF THE POWER TO THE SOFTENER.
- 7) TURN ON THE WATER TO THE SOFTENER.
- 8) ALLOW THE SOFTENER TO TAKE UP MOST OF THE BLEACH SOLUTION. TRY NOT ALLOW AIR TO GET INTO THE SOFTENER.
- 9) WHEN THE BLEACH CONTAINER IS ALMOST EMPTY, TURN OFF THE WATER TO THE WATER SOFTENER. ALLOW THE BLEACH TO DWELL FOR APPROXIMATELY THREE HOURS.
- 10) CONNECT THE BRINE TUBE TO A DRAIN.
- 11) TURN ON THE POWER TO THE CONTROLLER.
- 12) ADJUST THE TIMER TO THE CURRENT TIME OF DAY.
- 13) TURN ON THE WATER TO THE WATER SOFTENER.
- 14) SET THE TIME TO THE BRINE REFILL CYCLE (SEE MANUAL), AND ALLOW THE BRINE TUBE TO RINSE TO DRAIN. REPEAT THIS STEP UNTIL THE BRINE TUBE TEST NEGATIVE FOR BLEACH.
- 15) WHEN THE BRINE TUBE IS CLEAR OF BLEACH, RECONNECT IT TO THE AIR CHECK VALVE ON THE BRINE TANK.
- 16) SET THE TIMER TO THE BACKWASH MODE AND ALLOW THE SOFTENER TO RINSE TO DRAIN UNTIL THE DRAIN TESTS NEGATIVE FOR BLEACH. REPEAT THIS STEP IF NECESSARY.
- 17) WHEN ALL THE BLEACH IS RINSED FROM THE WATER SOFTENER, SET THE TIMER TO START THE REGENERATION CYCLE.